



# HAUSMANN INDUSTRIES, INC.

130 Union Street • Northvale NJ 07647 USA

Tel: (201) 767-0255 • Fax: (201) 767-1369 • [www.hausmann.com](http://www.hausmann.com)

## Model 2153 Air-Lift Stool



### Assembly Instruction


• Tools Required: No tools required for assembly.

• Parts Components

<u>PART</u>	<u>DESCRIPTION</u>	<u>QTY</u>
A-S	Stool Base	1
B-S	Stool Seat w/ Mounting Plate, Single Lever Adjustment	1
C-S	Plastic compressible Support Post Cover	1
D-S	Gas Lift Support Post	1
E-S	Friction Ring Stem Hooded Black 2" Twin Wheel Casters	5

# Model 2153 Air-Lift Stool

## Assembly Diagram

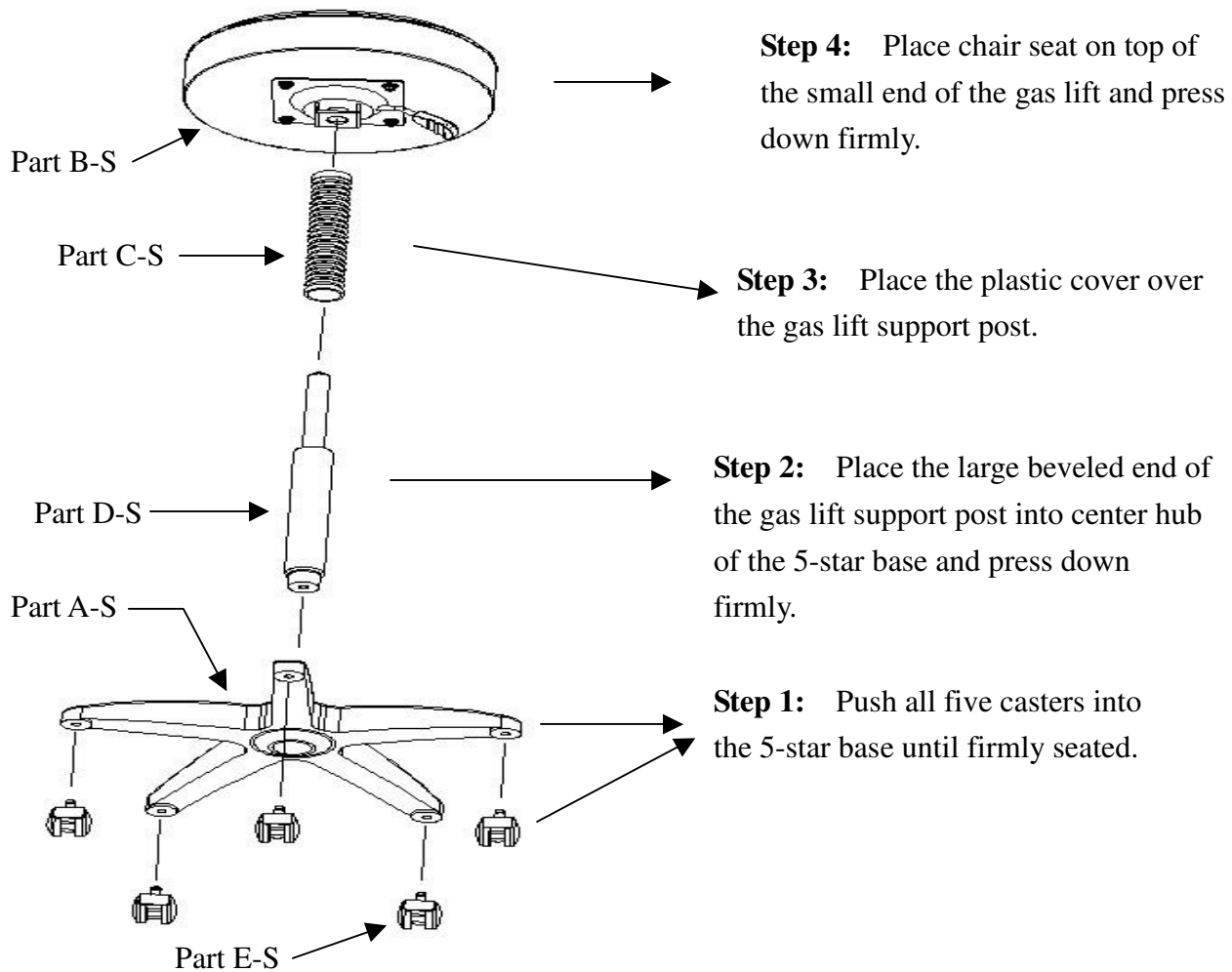

### Operation: (250 lbs. weight capacity)

1. To raise the seat, lift Hi-Lo Control Handle upward. To set the seat in the desired position release the Hi-Lo Control Handle.
2. To lower the seat, lift the Hi-Lo Control Handle upward while allowing your body weight to lower the seat into the desired position. To set the desired position release the Hi-Lo Control Handle.

**This stool is designed for Professional use by Doctor, Therapist, Nurse or Staff.**

**CAUTION: NOT FOR PATIENT USE.**

**WARNING:** This stool is designed to sit with all its legs or wheels in contact with the floor. Do not lean back on this stool in such a way as to bring one or more legs or wheels off the floor. Failure to follow this warning could result in serious injury.

### Troubleshooting Guidelines / Maintenance

- For assistance please call Hausmann Industries for service at (201) 767-0255, press 5 for Customer Service. We may request purchase details (date purchased, dealer) to better assist you.