SENNHEISER

EZT 3012

- DE Bedienungsanleitung
- EN Instruction manual
- FR Notice d'emploi
- ES Instrucciones de uso
- IT Istruzioni per l'uso
- NL Gebruiksaanwijzing

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NL

Important safety instructions

- ▷ Read this instruction manual carefully and completely before using the product.
- ▷ Make the instruction manual easily accessible to all users at all times.
- ▷ Always include the instruction manual when passing the product on to third parties.
- ▷ Do not use an obviously defective product.
- ▷ The product generates alternating magnetic fields during normal use that could cause interference with cardiac pacemakers, implanted defibrillators (ICDs), and other implants. Always maintain a distance of at least 3.94"/10 cm between the product and the cardiac pacemaker, implanted defibrillator, or other implant.
- ▷ Keep the product, accessories and packaging parts out of reach of children and pets to prevent accidents. Swallowing and choking hazard.
- ▷ Do not use the product in an environment that requires your special attention (e.g. when performing skilled jobs).
- ▷ Always keep the product dry and do not expose it to extreme temperatures to avoid corrosion or deformation.

DE

EN

FR

ES

IT

NL

Intended use/Liability

The EZT 3012 induction loop is an accessory for Sennheiser infra-red and RF receivers as well as for conference units of a conference system. The EZT 3012 inductively couples the audio signal from the receiver/conference unit to hearing aids.

It is considered improper use when the product is used for any application not named in the corresponding product documentation.

Sennheiser does not accept liability for damage arising from misuse or improper use of this product and its attachments/accessories.

Before putting the product into operation, please observe the respective country-specific regulations.

Package contents

1 EZT 3012 induction loop

1 instruction manual

Using the EZT 3012

CAUTION

Danger due to magnetic fields!

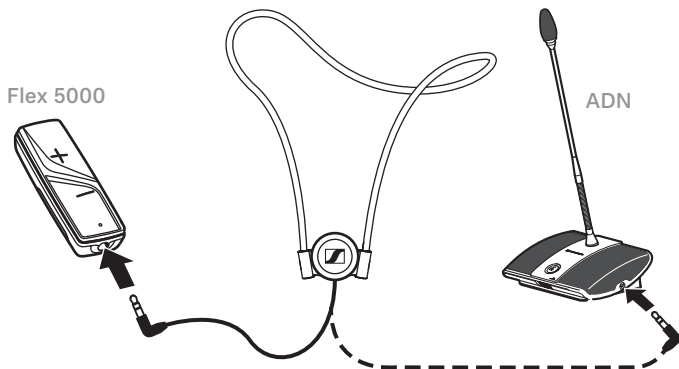
If you connect the EZT 3012 to devices that are not designed for use with the induction loop, magnetic fields can be generated that could cause dangerous interference with cardiac pacemakers, implanted defibrillators (ICDs), and other implants.

▷ Only connect the induction loop to Sennheiser devices.

Connecting the induction loop

- ▷ Connect the 3.5 mm jack plug of the induction loop to your receiver or to the corresponding audio output of your Sennheiser device.

- ▷ Set your hearing aid to the so-called T-position. If your hearing aid has no T-position, please contact your hearing aid acoustician.

i For alternative connection possibilities to your hearing aid as well as the connection to cochlea implants, please contact your hearing aid acoustician.

DE

EN

FR

ES

IT

NL

Listening with the induction loop

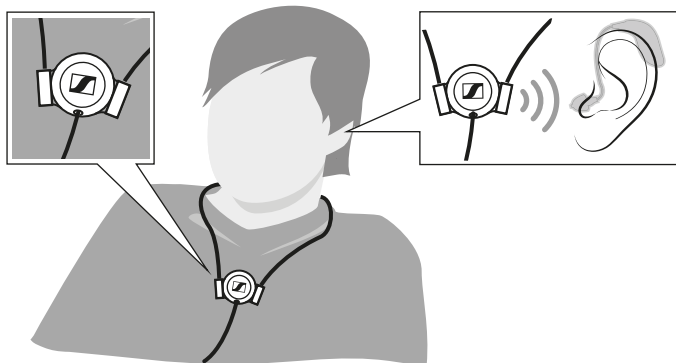
WARNING

Danger of hearing damage due to high volume levels!

Listening at high volume levels can lead to permanent hearing defects.

- ▷ Before using the receiver or the conference unit, set the volume to a low level.
 - ▷ Do not continuously expose yourself to high volume levels.
-
- ▷ Place the induction loop around your neck, with the large Sennheiser logo facing away from your body.
 - ▷ Switch your audio source on.
 - ▷ Switch the receiver or the conference unit on.
 - ▷ The easy, individual adjustment of the sound to your personal hearing ability can be set, for example, via the volume control on the receiver. This allows hearing aid users to enjoy an undisturbed listening experience in optimum sound quality, without the need for cables.

DE

EN

FR

ES

IT

NL

Cleaning and maintaining the EZT 3012

- ▷ Clean the induction loop with a slightly damp cloth.

Specifications

EZT 3012 induction loop

Audio connection	3.5 mm stereo jack plug
------------------	-------------------------

Cable length	
--------------	--

Induction loop	approx. 75 cm
----------------	---------------

Connection cable	approx. 40 cm
------------------	---------------

Impedance	28 Ω at 1 kHz
-----------	----------------------

Manufacturer declarations

For more information on regulatory compliance markings, see the inside back cover.

Warranty

Sennheiser electronic GmbH & Co. KG gives a warranty of 24 months on this product. For the current warranty conditions, please visit our website at www.sennheiser.com or contact your Sennheiser partner.

For AUSTRALIA and NEW ZEALAND ONLY

Sennheiser Australia Pty Ltd provides a warranty of 24 months on these products. For the current warranty conditions, visit the Sennheiser website:

- Australia: <https://en-au.sennheiser.com>
- New Zealand: <https://en-nz.sennheiser.com>

Sennheiser goods come with guarantees that cannot be excluded under Australian and New Zealand Consumer law. You are entitled to a replacement or refund for a major failure and compensation for any other reasonably foreseeable loss or damage. You are also entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and the failure does not amount to a major failure.

This warranty is in addition to other rights or remedies under law. Nothing in this warranty excludes, limits or modifies any remedy available to be consumer which is granted by law.

To make a claim under this contract, raise a case via Sennheiser website:

- Australia: <https://en-au.sennheiser.com/service-support-contact>

- New Zealand: <https://en-nz.sennheiser.com/service-support-contact>

All expenses of claiming the warranty will be borne by the person making the claim.

Sennheiser international warranty is provided by:
Sennheiser Australia Pty Ltd (ABN 68 165 388 312) Level 14,
Tower A Zenith Building, 821 Pacific Highway, Chatswood
NSW 2067, Australia.

EU declaration of conformity

- RoHS Directive (2011/65/EU)

The full text of the EU declaration of conformity is available at the following Internet address:
www.sennheiser.com/download.

UK declaration of conformity

- RoHS Regulations (2012)

Notes on disposal

- EU: WEEE Directive (2012/19/EU)
- UK: WEEE Regulations (2013)

The symbol of the crossed-out wheeled bin on the product, the battery/rechargeable battery (if applicable) and/or the packaging indicates that these products must not be disposed of with normal household waste, but must be disposed of separately at the end of their operational lifetime. For packaging disposal, please observe the legal regulations on waste segregation applicable in your country.

DE

EN

FR

ES

IT

NL

Further information on the recycling of these products can be obtained from your municipal administration, from the municipal collection points, or from your Sennheiser partner.

The separate collection of waste electrical and electronic equipment, batteries/rechargeable batteries (if applicable) and packagings is used to promote the reuse and recycling and to prevent negative effects caused by e.g. potentially hazardous substances contained in these products. Herewith you make an important contribution to the protection of the environment and public health.

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Sennheiser is a trademark of Sennheiser electronic GmbH & Co. KG, registered in the U.S. and other countries.
Other product and company names mentioned in the user documentation may be the trademarks or registered trademarks of their respective holders.