

# BIOS living

## Overbed Table

### LF844

#### Technical Features

- Table dimensions: 30¼" x 15" / 77 x 38 cm
- Weight: 20 lbs / 9 kg
- Max. Height: 30" - 47¼" / 76 - 120 cm
- Material: Steel based epoxy, chromed column, clear laminated beech wood table

#### WARNING

The overbed table is not designed to support heavy weight.

**DO NOT** lean on the table for support when getting up from a seated position.

**Maximum load capacity: 22 lbs / 10 kg**

#### Assembly Instructions

- Remove the protective film covering the table and on the 4 wheels.
- Place post into slot at base of the table using 6 x 20 mm screws. **(fig 1)**
- Attach the post to the base using 2 x 50mm screws **(fig 2)** also using screw enforcers **(fig 3)** and washers.
- Unscrew the security pin in order to unlock the column **(fig 4)**, adjust the table to desired height.

#### Parts Included

- 1 table
- 1 adjustable post
- 1 steel base and 4 wheels (with brakes)
- 2 50 mm screws
- 2 screw enforcers
- 2 washers
- 6 20 mm screws
- 1 screw driver



## Operating Instructions

- **(fig 5)** Shows the overbed table fully assembled. The table height can be adjusted by squeezing the ratchet lever towards the table top (fig 6). Using your other hand support the other end of the table top while adjusting the height.



## Recommendation

- Please test the stability of your table before using it for the first time.
- **Do not** suddenly apply great pressure on your table.
- **Do not** apply more than 22 lbs / 10 kg of pressure on your table.

## Regular Maintenance

- Remove and wipe any spills immediately in order to avoid any warping of the table.
- Use soap and water to remove film and dirt and dry completely.
- **Do not** use abrasive cleaning products such as bleach which will damage your table and cause discoloring.
- Only a licensed technician should be contacted to make any repairs and or replacements of parts.

## One Year Limited Warranty

The Overbed Table has a 1 year warranty to be free of manufacturing defects in materials and workmanship under normal applications for 1 year of the original owner. If this product becomes inoperable due to defect and requires repair, return the product with all component pieces and proof of purchase to the address listed below. This warranty does not cover any shipping/transport costs. This warranty does not apply if the product is subject to misuse, neglect, rough handling or damage.

Ship the unit prepaid and insured (at owners option) to:

### **THERMOR LTD.**

Attn: Repair Department  
16975 Leslie Street  
Newmarket, ON L3Y 9A1

www.biosmedical.com  
Email: support@biosmedical.com

BIOS | Medical  
MANUFACTURED BY / FABRIQUÉ PAR :  
**THERMOR LTD.**  
16975 LESLIE STREET  
NEWMARKET, ON L3Y 9A1  
MADE IN CHINA / FABRIQUÉ EN CHINE  
WWW.BIOSMEDICAL.COM