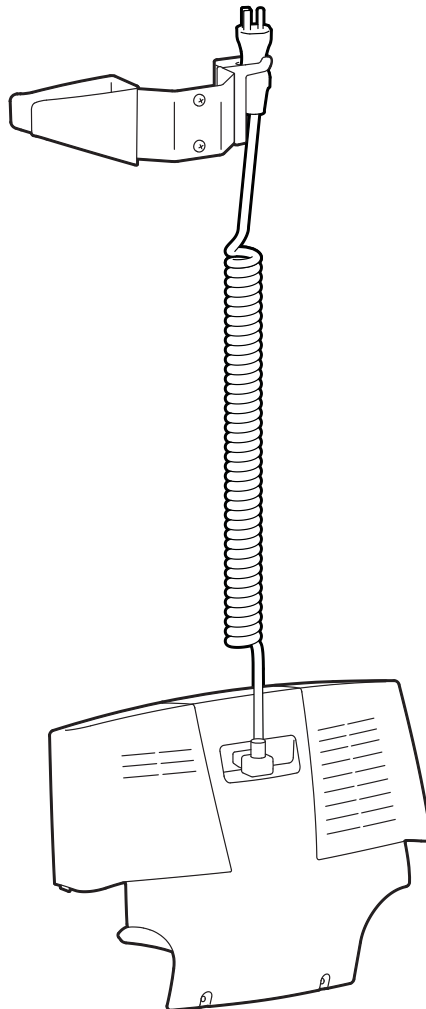
# ergotron®

---

## StyleView™ Power Cord Upgrade Kit for LiFe Carts

---

**EN** User Guide

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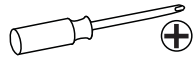
[www.ergotron.com](http://www.ergotron.com) | USA: 1-800-888-8458 | Europe: +31 (0)33-45 45 600 | China: 400-120-3051 | Japan: [japansupport@ergotron.com](mailto:japansupport@ergotron.com)

---

Tools Needed

	A	B	C	D
1	1x		1x	
2			1x	
3	1x	1x	1x	2x

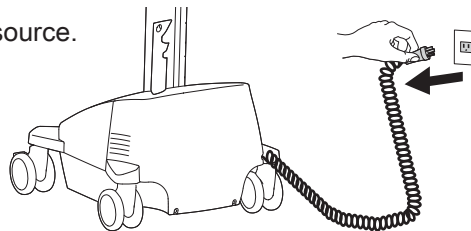
1

Turn off all mounted equipment.


2

Disconnect Power System from power source.



3

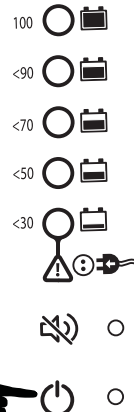Turn power system off by holding down the AC Outlet Power button for 1 - 3 seconds. Power light will shut off.



4

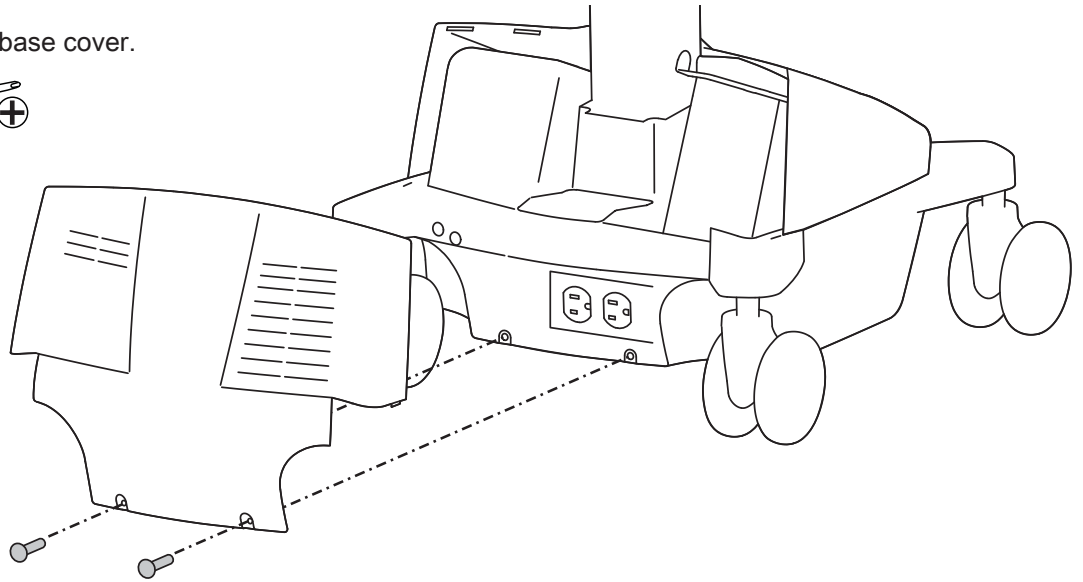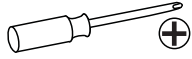Remove rear base cover.


5

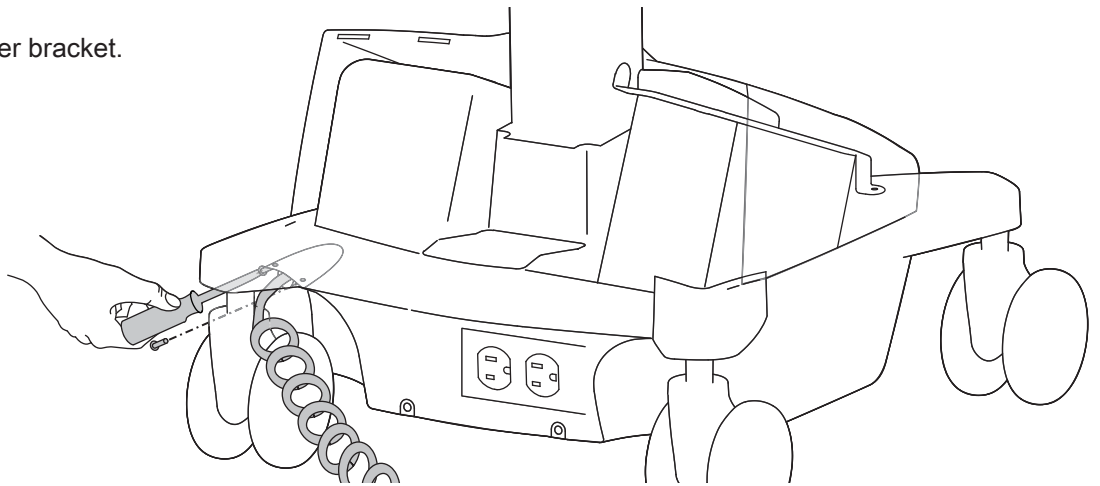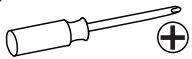Remove retainer bracket.

6

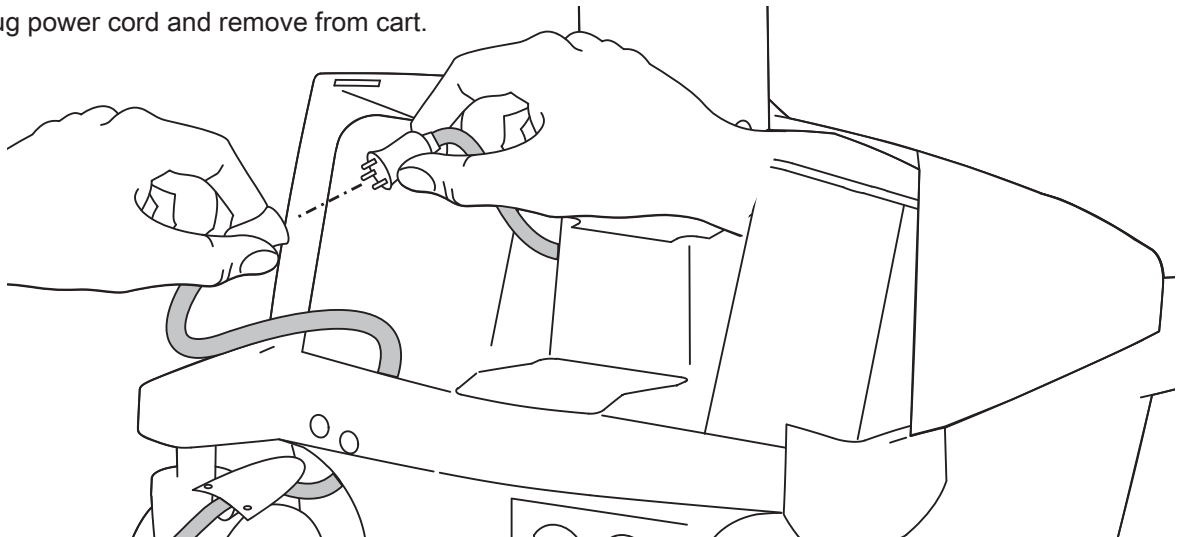
Unplug power cord and remove from cart.


# 7

Attach new power box.

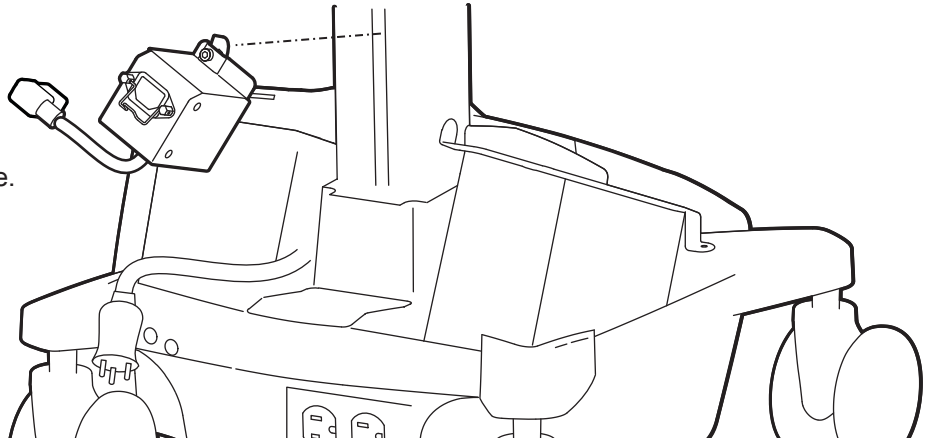
1 Insert bracket into T-slot at an angle.

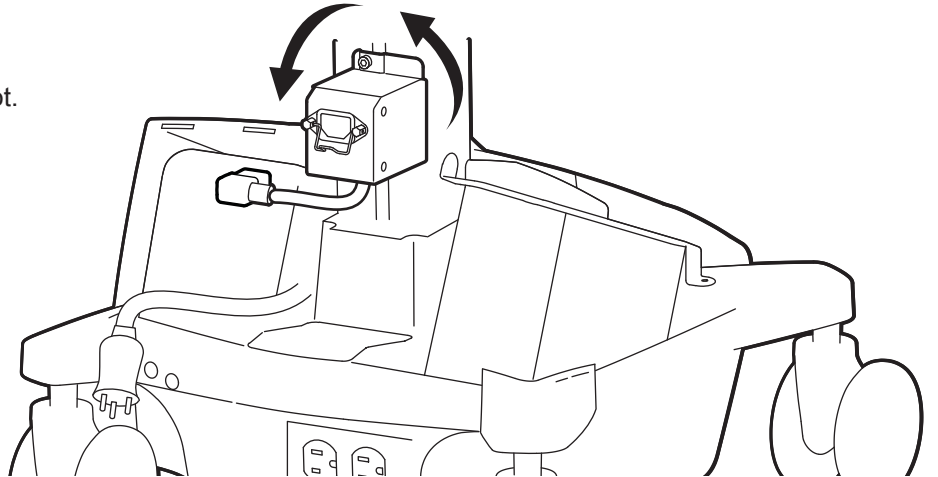
2 Rotate so bracket catches inside slot.

3 Slide down until it stops.

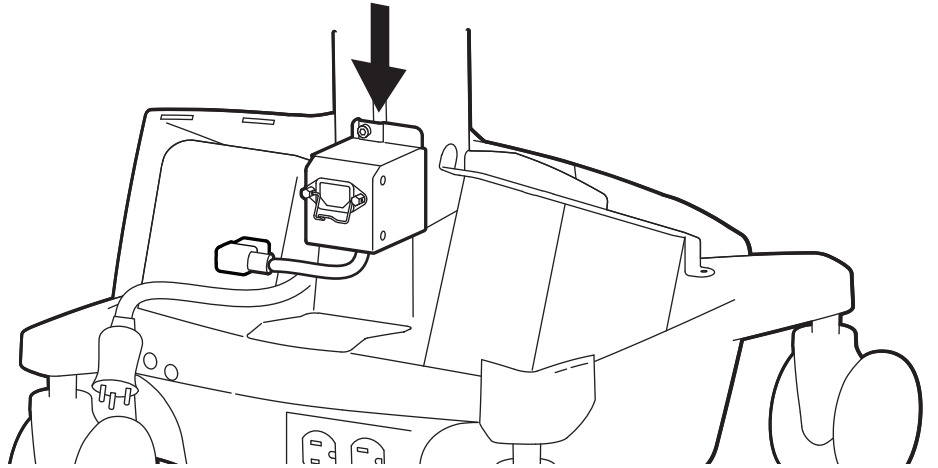
4 Tighten to secure.



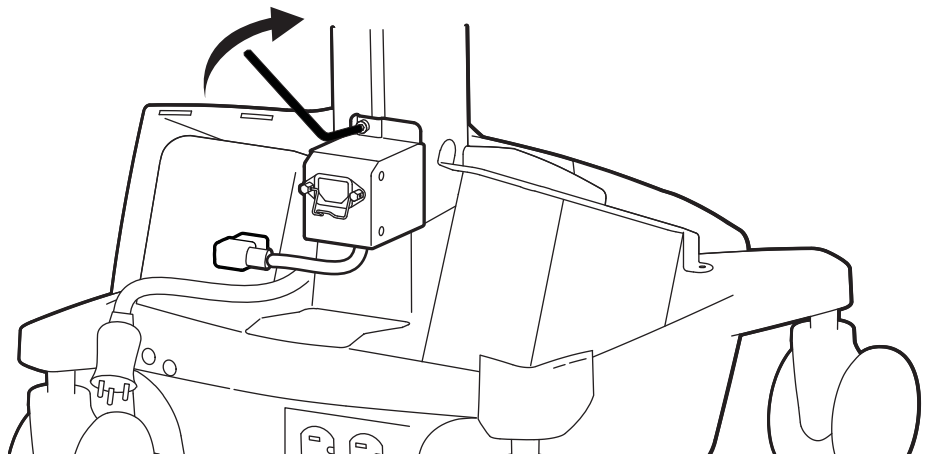
8

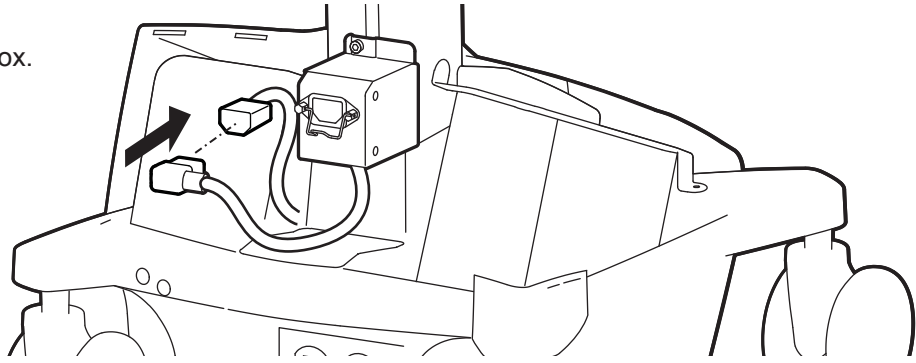Plug in power box.




9

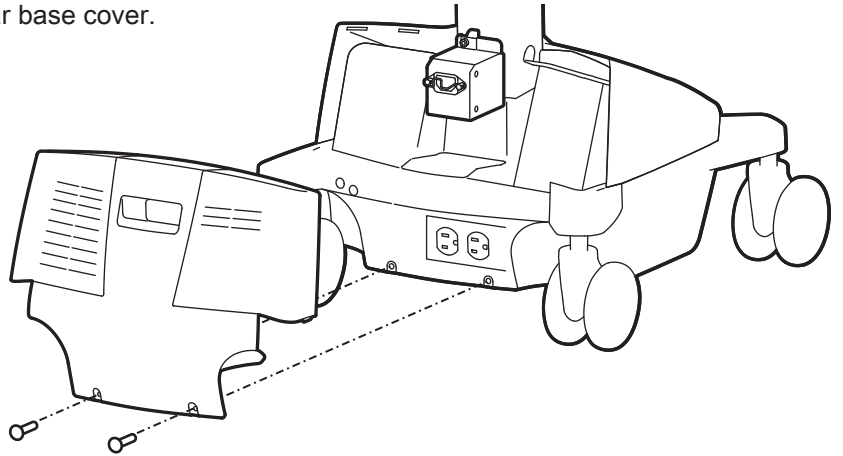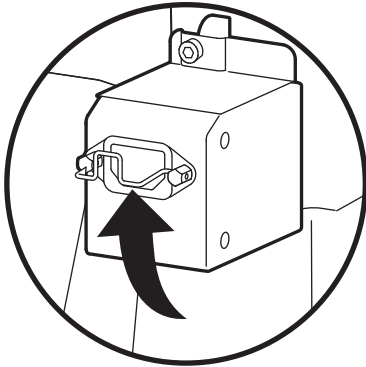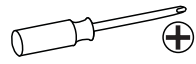Flip retainer up then attach new rear base cover.


10

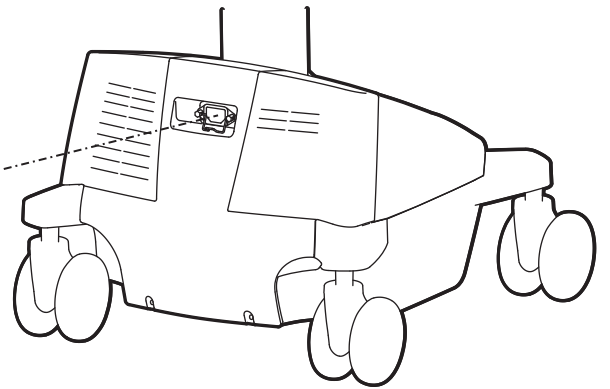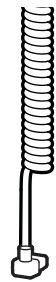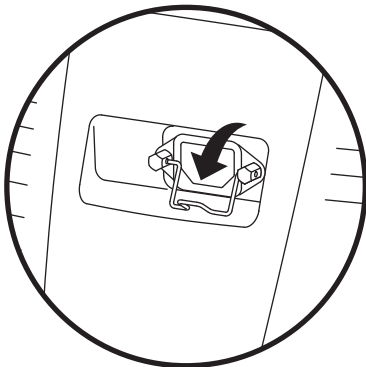Flip retainer down, hang power cord on hook and plug into base.



11



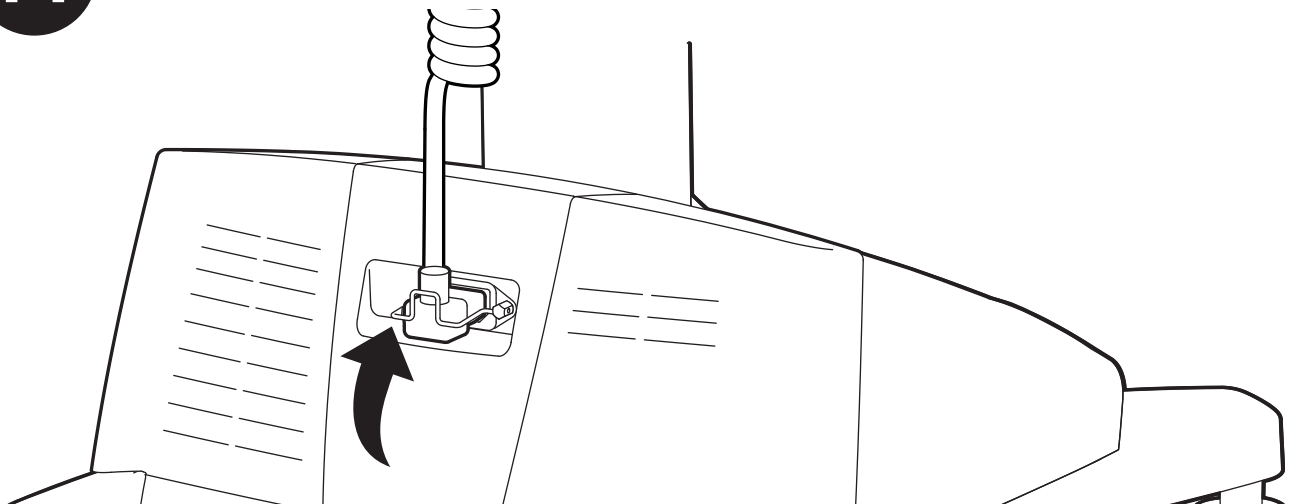
Flip retainer up to secure power cord.

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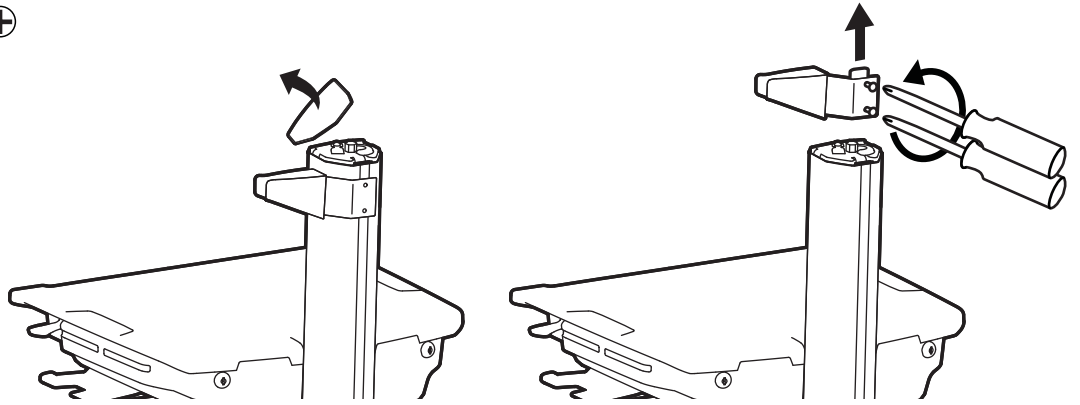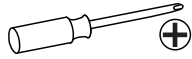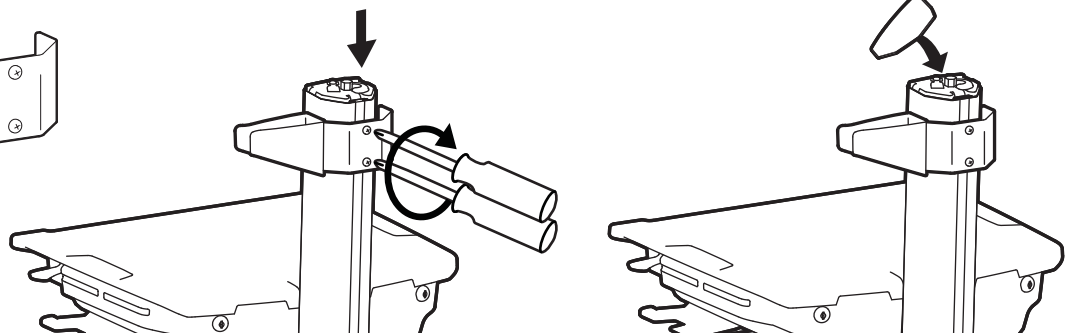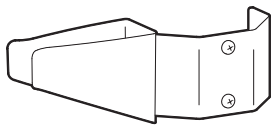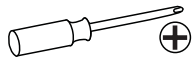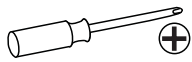
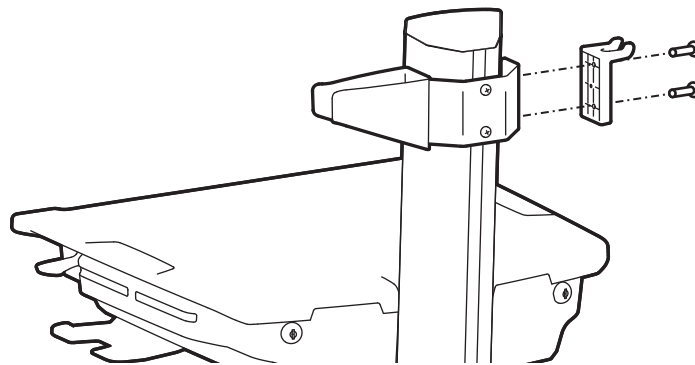
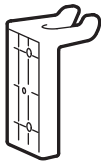
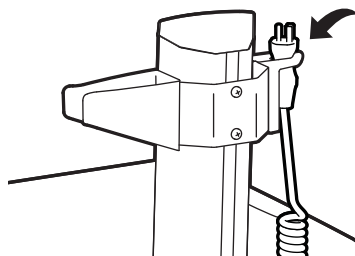
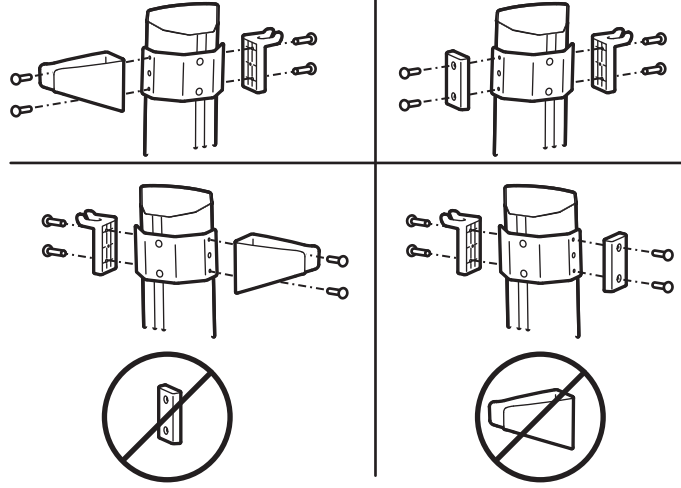
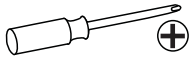
© 2021 Ergotron, Inc. All rights reserved. StyleView™ is a registered trademark of Ergotron, Inc.

While Ergotron, Inc. makes every effort to provide accurate and complete information on the installation and use of its products, it will not be held liable for any editorial errors or omissions (including those made in the process of translation from English to another language), or for incidental, special or consequential damages of any nature resulting from furnishing this instruction and performance of equipment in connection with this instruction. Ergotron, Inc. reserves the right to make changes in the product design and/or product documentation without notification to its users. For the most current product information, or to know if this document is available in languages other than those herein, please contact Ergotron. No part of this publication may be reproduced, stored in a retrieval system, or transmitted in any form or by any means, electronic, mechanical, photocopying, recording or otherwise without the prior written consent of Ergotron, Inc., 1181 Trapp Road, Eagan, Minnesota, 55121, USA Patents Pending and Patented U.S. & Foreign. Ergotron is a registered trademark of Ergotron, Inc.

888-98-206-W-00 rev. A • 06/21

7 of 7